

# InterDry<sup>®</sup>

*Moisture-wicking fabric with antimicrobial silver*

Innovative and effective therapy for the treatment of intertrigo, candidiasis, erythema and other skin complications.<sup>1,2,3</sup>



## References

1. Vorbeck, E. Evaluation of a Skin Fold Management Textile with Antimicrobial Silver Complex in a Variety of Case Studies
2. A Case Series Using A Polyurethane Coated Polyester Textile Impregnated with An Antimicrobial Silver Complex for The Management Of Maceration, Odor and Redness In Skin Folds. The Experience From A Wound Healing and Lymphedema Management Clinic. Erik Maus MD, CWS; Dennise Baylor CLT; Susan Benavides CLT; Lisa Kinder CLT; Caroline E. Fife MD; Renie Guillod MD; Trina Hawkins ANP; Michelle Robles CLT; Latisha A Smith MD; Grisseld Virtute CLT
3. Multi-site Feasibility Study Using A New Textile with Silver for Management of Skin Conditions Located In Skin Folds. Theresa Henn, RN-CS, BSN, G/ANP, TLC, HealthCare Company, Tucson, AZ; Karen Lou Kennedy-Evans, RN, CS, FNP, Wound and Skin Care Consultant, Tucson AZ

## Moisture-wicking fabric with antimicrobial silver

### Uses


- Manages moisture, friction and bacterial or fungal organisms in skin folds<sup>1</sup>
- Manages the symptoms associated with intertrigo such as erythema, painful rash, denudement, satellite lesions and odor<sup>2</sup>

### How to use


Refer to product package for complete instructions including directions for use, indications, precautions, warnings, storage, and other information. If skin irritation does not improve or worsens over time, talk to a physician or other healthcare professional.



1. Wash skin gently. Pat dry, do not rub.

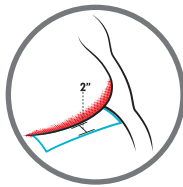


4. Leave at least 2 inches of the fabric exposed outside the skin fold.

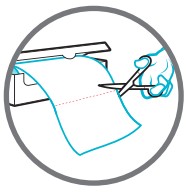


2. With clean scissors, cut enough fabric to cover the affected area, allowing for a minimum of 2 inches to extend beyond the skin fold for moisture evaporation.



5. Secure the fabric in one of several ways: with the skin fold, with a small amount of skin-friendly tape, or tucked under clothing.

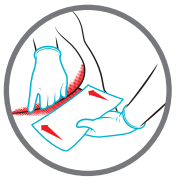

3. Lay a single layer of fabric in the skin fold, placing one edge into the base of the fold. Gently smooth the rest of the fabric over the skin, keeping it flat.

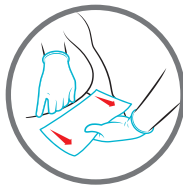

6. Remove the fabric before bathing and reuse it when finished. When removing, gently separate the skin fold and lift away.



SKIN IRRITATION IN SKIN FOLDS, INTERTRIGO

### Helpful hints

- InterDry® can be written on with a ballpoint pen. It may be helpful to label InterDry with the date it was placed on the patient, or follow your facility's protocols
- Each piece of InterDry may be used up to 5 days, depending on fabric soiling, odor, amount of moisture and general skin condition. Replace InterDry if it becomes soiled with blood, urine or stool
- Do not use creams, ointments or powders with InterDry as it may interfere with product efficacy

### ORDERING INFORMATION



PRODUCT	SIZE	UNITS
7910	10 x 144 in.	10/case
7912	10 x 36 in.	10 pks/box



Textile with Silver

### References

1. Vorbeck, E. Evaluation of a Skin Fold Management Textile with Antimicrobial Silver Complex in a Variety of Case Studies
2. Freyberg, J, et al. Moisture Management Challenges for the WOC Nurse presented at the WOCN Society 39th Annual Conference, June 2007

Important Safety Information: InterDry® is a skin protectant for the management of skin folds and other skin-to-skin contact areas. It reduces microbial colonization in the fabric. This single use fabric provides moisture translocation to keep skin dry while the antimicrobial in the fabric reduces odor. The safe use of InterDry fabric during pregnancy, lactation and on children has not been demonstrated. Although many patients benefit from the use of this product, results may vary. Discuss the treatment options with your physician to understand the risks and benefits to determine if InterDry is right for you. For further questions, call Coloplast Corp at 1-800-533-0464 and/or consult the company website at [www.coloplast.us](http://www.coloplast.us).

Ostomy Care / Continence Care / Wound & Skin Care / Urology Care